## ENGLISH

## Device Description

The product is a mobility device that partially or fully supports the weight of the patient. The body weight support is adjustable according to the user's needs, so that they may practice at their personal limits.

## Intended Use

The device is intended to provide adjustable weight support for people with walking disabilities who lack strength, good balance, or endurance.

## Technical information

The target group is based on functional abilities and anthropometry.

User's Dimensions	
Min. Crotch Height cm (in)	32 (12,5)
Max. Crotch Height cm (in)	56 (22)
Min. Arm pit Height cm (in)	60 (23,5)
Max weight kg (lbs)	45 (100)

Make sure that the Levity is appropriate for the user before proceeding with the installation. Consult the user's physiotherapist or health specialist to confirm the compatibility and safety.

## Contraindications

The device might be contraindicated for patients having recent wounds located near the harness area or those who are living with a condition causing irritability of the skin.

## Adverse Effects

Discomfort, Temporary skin rash, Minor bleeding, Temporary injury, Permanent injury.

## Storage

The Levity's safe operating and storage temperature is 5 to 40° C. Ensure that the Levity's harness is dry before use.

## Cleaning and maintenance

- The device's metal and plastic components can be cleaned with a reasonable amount of soapy water or a non-abrasive detergent to prevent the degradation of components. The fabrics can be cleaned by hand on the device or removed and laundered with non-abrasive detergent.

- Knobs, brakes, levers, clips, straps, covers, and fasteners, must be inspected before each use. The device's structural integrity should also be verified before each use.

If cracks, excessive wear, excessive deformations, general instability, or missing components are detected, the product must be immediately removed from use and Ora Medical should be contacted.

## Warnings

- Adult supervision is required at all times.
- The device is designed for daily use not exceeding four hours a day.
- The Levity is intended for indoor use only. Do not use the Levity on rough or uneven terrain. Never use the Levity near stairs or swimming pools.
- Users should be dressed when using the device.
- Never use a broken or damaged Levity.
- Any unauthorized modification to the Levity may affect the user's safety and result in serious injury or death.
- The user's weight must be supported by the harness seat only. The harness belt must only prevent the user from losing balance.
- Keep fingers clear of all moving parts when using, folding or adjusting the Levity.
- The selection of the weight-bearing balance module and components such as the lower body support must be made with the help of a professional care giver or health specialist to make sure it's well adjusted to the patient's weight and physical ability.
- When adjusting the harness width, leave at least 2,5 cm (1 inch) of rail when tightening the fasteners. Use the indicator sticker to make sure the harness is sufficiently inserted in the rail.
- The Levity's wheels and carriage brake lever should always be locked when the device is not used and when no weight-bearing module is in place.
- All straps, levers, and other fastening elements must be securely fastened before use or transport.
- Ensure the brakes are locked and no user is installed in the harness before adjusting the weight-bearing mechanism.
- The access cover must be closed at all times with the exception of changing the weight-bearing module.
- For storage and transport ensure the harness is at its highest position and that the harness brakes are engaged. Detach the weight-bearing module and ensure the Levity's wheels are locked.
- Ensure that the straps are sufficiently clear of the user's neck when using the Levity to avoid any risk of strangulation.
- In case of any serious incident that has occurred in relation to the device, the user and/or patient should report the incident to Ora Medical and the Therapeutic Goods Administration or EU competent authority of the Member State in which the user and/or patient is established.

## Device Lifetime and disposal

After a lifetime of 3 years, when disposing of the Levity using a community waste disposal site, the frame and any accessory components should be disposed separately to make sure the device cannot be used by others.

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